

Short form instructions

Radiometer **ABL90 FLEX PLUS** with syringe samples




1. If using safePICO syringe:
If not, then go to step 2
Place syringe into the automatic sample mixer



2. Ensure analyser is in **Ready** mode and desired parameters are in green



3. Logon to the analyser
4. Tap the **Syringe** button and inlet will open

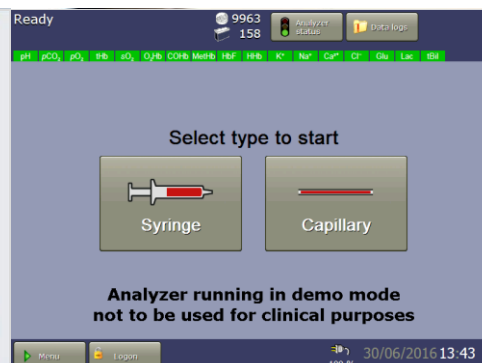


5. **Align** syringe with inlet gasket

6. **Push** the syringe onto the inlet



7. **Hold** syringe in place while the sample is aspirated
8. When prompted gently **remove** the syringe, the inlet will close automatically

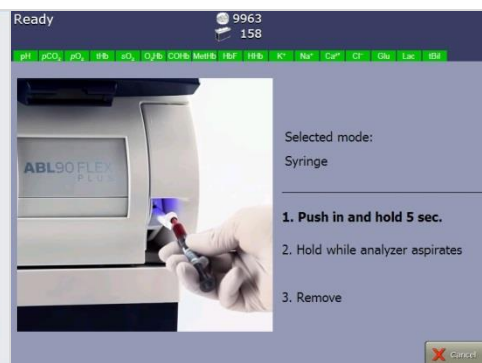

9. Enter appropriate patient information when prompted

Wait for results



Done!